

# User Manual

## ROM Knee Brace



## How to put on the knee brace




1. Loosen all straps before putting on the knee brace.



2. Position the angle adjuster at the center of the knee.



3. Fasten the 6 straps in sequence.

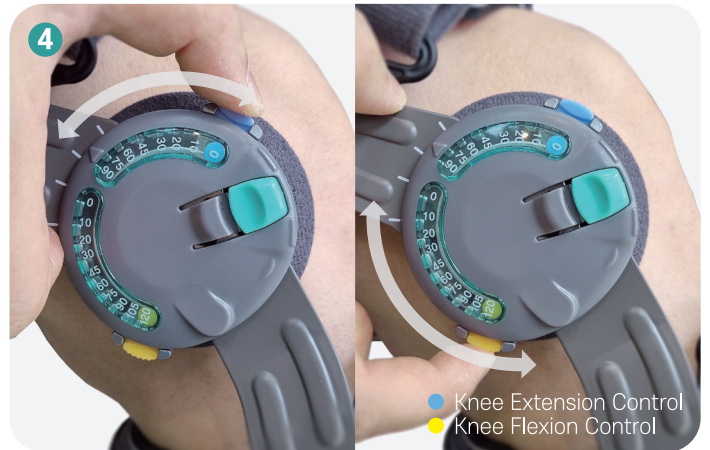

Press the button to adjust the angle.  
Blue button for extension and yellow button to limit the range of flexion.


## How to adjust strap length



1. Detach the strap that needs to be adjusted.



2. Separate the Velcro at the end of the strap.

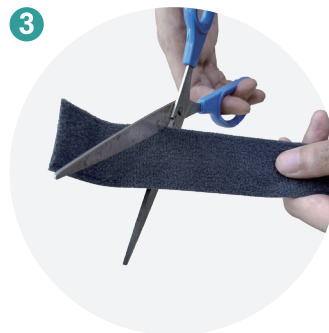

3. Cut the strap to the desired length.




4. Finish by reattaching the Velcro

## Easy angle locking



1. Used to immobilize the knee for a period of time.

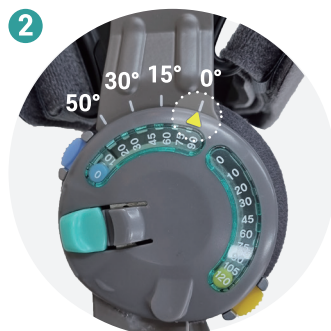

2. Align the arrowhead with the white line.



3. Slide the green button to Lock.



4. To unlock, press the protruding part while sliding the button to release it.